**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61 2 9890 8696  
 F +61 2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43-512 57 95 15  
 F +43-512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

4 051526 949508

medi Brasil  
 Rua Cristovan de Vita, n.º 260, Un. 30,  
 Centro Logístico Raposo Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45 70 25 56 10  
 F +45 70 25 56 20  
 kundeservice@sw.dk  
 www.medidanmark.dk


E013733/06.2020

**medi**

**circaid® power added compression™  
 (pac band™)  
 inelastic compression system**

Instructions for use. Gebrauchsanweisung.  
 دليل الاستخدام. Návod k použití. הוראות שימוש.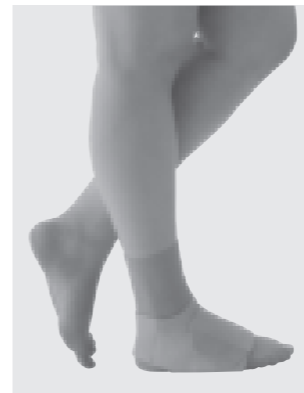

**power added compression™  
 (pac band™)**

**medi. I feel better.**

medi Bayreuth España SL  
 C/Canigó 2 – 6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34 93 260 04 00  
 F +34 93 260 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33 1 48 61 76 10  
 F +33 1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1371 0090  
 F +36 1371 0091  
 info@medi.hu  
 www.medi.hu

medi Italia s.r.l.  
 Via Giuseppe Ghedini, 2  
 40069 Zola Predosa (BO)  
 Italy  
 T: +39-051 6 13 24 84  
 F: +39-051 6 13 29 56  
 info@medi-italia.it  
 www.medi-italia.it






medi Japan K.K.  
 5-1 Nihonbashi Kabutocho Chuoku  
 103-0026 Tokyo  
 Japan  
 T: +81 3 6778 2590  
 F: +81 3 5847 7901  
 info@medi-japan.jp  
 www.medi-japan.co.jp

Tachezy Sanit Srl.  
 Via Druso 48/a  
 39100 Bolzano  
 Italy  
 T: +39 0471 288 068  
 F: +39 0471 282 773  
 info@tachezysanit.com  
 www.tachezysanit.com

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31 76 57 22 555  
 F +31 76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Norway AS  
 Vestheimvegen 35  
 4250 Kopervik  
 Norway  
 T: +47 52 84 45 00  
 F: +47 52 84 45 09  
 post@medinorway.no  
 www.medinorway.no

English

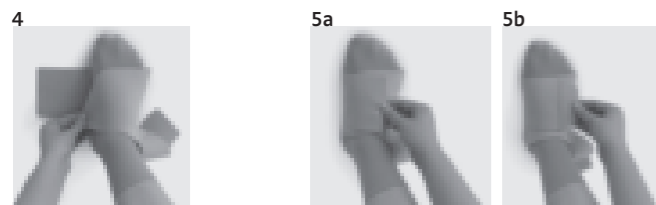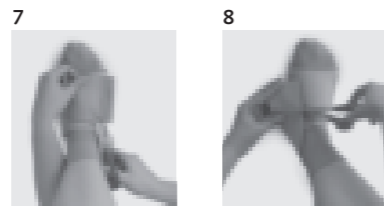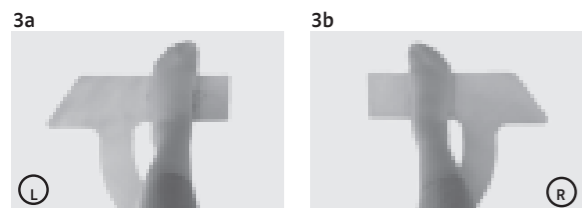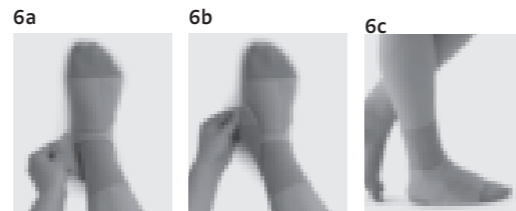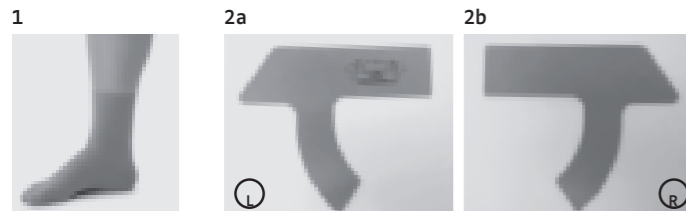## circaid® power added compression™ (pac band™)

### Purpose

The garment is designed to provide compression to the foot and ankle region for patients with venous and lymphatic disorders. It is intended to be used in conjunction with the circaid compression ankle, however can also be used as a standalone product as well.

### Indications

- Lymphedema
- other forms of edema
- Lipedema
- Chronic venous insufficiency
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis

### Contraindications

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

### Cautions

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)
- Pediatric & assisted use - must be able to sense & communicate applied pressure

### Storage instructions

Store in a dry place

Keep out of sunlight

medi Polska Sp. z o.o.  
ul. Łąbiedzka 22  
44-121 Gliwice  
Poland  
T: +48-32 230 60 21  
F: +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
Rua do Centro Cultural, no. 43  
1700-106 Lisbon  
Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LLC  
Business Center NEO GEO  
Butlerova Street 17  
117342 Moscow  
Russia  
T +7-495 374 04 56  
F +7-495 374 04 56  
info@medirus.ru  
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND  
CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

medi Sweden AB  
Box 6034  
192 06 Sollentuna  
Sweden  
T +46 8 96 97 98  
F +46 8 626 68 70  
info@medi.se  
www.medi.se

Medi Turk Ortopedi Medikal Iç ve  
Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak  
No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 0EL  
Great Britain  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

## Lifespan

Due to material wear and tear the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, doffing).

circaid power added compression band: <3 month - depends on the wear and tear

## Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

## Material composition

Nylon 74%  
Polyuretane 21%  
Elastane 5%

This garment does not contain latex.

## Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to keep the Velcro tab material lint-free and to protect other clothing from getting damaged. To extend the life of the garment, hand wash and drip dry.

- 
-  Machine wash warm
  -  Do not bleach
  -  Drip dry or line dry
  -  Do not iron
  -  Do not dry clean
  -  Do not wring
- 

## Wearing recommendation:

The circaid compression garment should never hurt. It should feel firm but comfortable. Slightly loosen the bands of the garment for night-time wear. If experiencing any pain, remove the garment and consult your physician.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands. The Velcro tabs should never touch the skin!

### **Donning**

The circaid power added compression band (pac band) is designed to provide additional compression to the foot, targeting the arch and ankle area regions.

For complete compression of the foot, it is recommended to apply the circaid power added compression band over the circaid compression ankle.

Step 1: Lay the pac band, out flat. If using on the left foot, the care instructions and illustration should be visible (2a). If using on the right foot, no text should be visible (2b).

Step 2: Center the arch of the foot as shown in the illustration

printed on the band. (3a) shows the left foot. (3b) shows the right foot. Wrap the short arch band over the top of the foot (4).

Step 3: Secure the long arch band over the top of the foot with the large hook tab (5a-b).

Step 4: Pull the ankle band around the back of the ankle and secure it with the small hook tab to a firm but comfortable compression (6a-c).

### **Note:**

If excess material or overlap is a concern, either the long arch band (7) or the ankle band (8) may be trimmed as needed. Be careful not to over-trim the bands.

### **Doffing instructions**

Disengage the hook tabs and fold each band back onto the body of the pac band. Do not roll the bands too tightly as this will cause the hook material to bend, reducing its adherence strength.

When complete, the garment should be ready for easy reapplication.

# circaid® power added compression™ (pac band™)

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circaid power added compression band:

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